



# How to use **BTNX Rapid COVID-19** antigen tests

## Detect **COVID-19**

### 1 SET UP

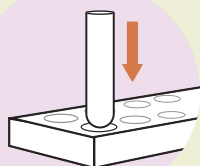


Place the tube in the holder

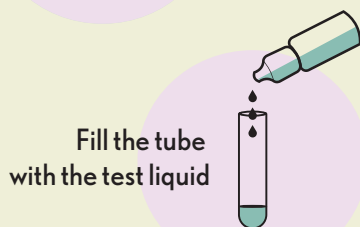


Fill the tube with the test liquid

### 2 COLLECTION

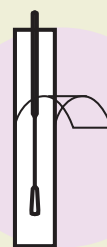

Peel down and remove nasal swab

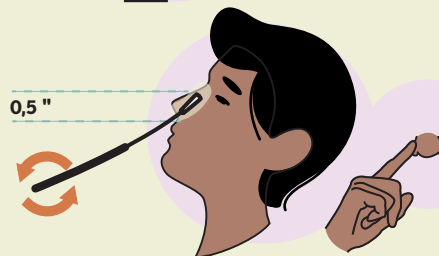

Compress the nostril with your finger as you turn the nasal swab


Insert nasal swab up to **0,5 inch** from edge of nostril. Roll **5 times** in each nostril; right & left

### 3 PREPARATION

Insert the swab in the tube liquid and rotate it **10 times**



Place the tube and swab in the holder and leave it there for **2 minutes**

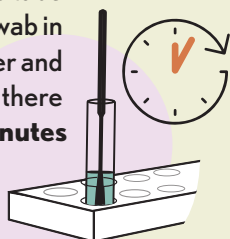

Remove the swab by pinching the tube



Close the tube with the stopper cap


### 4 TESTING

Place **3 drops** in the rapid response cartridge and wait **15 minutes**



(Do not touch it during the waiting time)

Here is an example of a positive test, negative and an invalid test

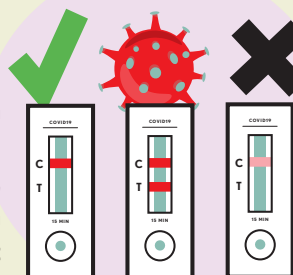

**NEGATIVE POSITIVE INVALID**

Redo the nasal swab with a new kit.

**☀ If you are positive, please call the Info-Covid line: 1-888-662-7482**

